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1. GENERAL INFORMATION

1.1 AIM AND CONTENTS
The aim of this manual is to supply all the information necessary so that the client, apart from attaining an adequate use of the appliance, will also be capable of using the instrument in the most autonomous and secure way possible.
1.2 CONSERVATION OF THE INSTRUCTION MANUAL
The instruction and maintenance manual must be kept with the product, inside the specially provided container and above all, away from any substances or liquids which could compromise perfect legibility.

2. AVERTENZE

2.1 GENERAL WARNINGS
Before carrying out any kind of operation on the product, the operator must carefully read the enclosed instructions, paying particular attention to the correct safety precautions and to the procedures to be followed for installation and for correct use.

3. DESCRIPTION OF PRODUCT

3.1 INTENDED USE
Aneroid sphygmomanometers allow for non invasive measurement of the arterial blood pressure, using an inflatable cuff.
3.2 MAIN COMPONENTS
The sphygmomanometer is made out of the following components:

4. OPERATING INSTRUCTIONS

4.1 TRANSPORT AND STORAGE
Before transporting the appliance, make sure it is correctly packaged ensuring also that there are no risks of shocks or falls during the transport itself.
4.2 PREPARATION
On receiving the product:

5. MAINTENANCE

5.1 CLEANING
Gauge
The check-up, to be performed before every use, must include a general control of the functioning of all parts of the device.